

# BeneVision NSeries 12 Lead ECG Quick Guide



**Please Note:** N1/ MPM/ T1 modules that support resting 12 lead ECG analysis display a 12-lead label on the side of the module.  Not all devices support this feature.

To enter the 12-Lead screen, follow this procedure:

1. Select the ECG numeric area or waveform area to enter the **ECG** menu.
2. From the bottom of the **ECG** menu, select **12-Lead**.

OR

1. Select the **ECG Full Screen** Quick Key (if applicable).



## Initiating Resting 12-Lead ECG Analysis

Before 12-lead ECG interpretation, check that all electrodes are correctly connected to the lead wires and the ECG trunk cable is properly connected. Check electrode placement and patient information is correct, keep the patient still.

To initiate 12-Lead ECG analysis, select **Analyse** from the left bottom of the 12-Lead screen (if using a T1 module, please wait 9 seconds to acquire the necessary 12 Lead ECG data before pressing analyse).



These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate Operator's Manual for detailed instructions. The Illustrations may not necessarily reflect the setup or data displayed on your device.

## Changing 12-Lead ECG Analysis Settings

On the ECG 12-Lead screen, select **Setup** to set the high frequency filter, baseline drift removal (BDR) switch, and the waveform layout.

**Please Note:** If the V4R electrode position is used when acquiring a paediatric ECG, V4R should be selected from the V3 placement options in the setup menu.

## Setting the 12-Lead Interpretation Report

To set the 12-lead interpretation report, follow this procedure:

1. On the ECG 12-Lead screen, select **Setup** to enter the **12-Lead Setup** menu.
2. Select the **Report** tab.
3. Set the format and items included in the 12-lead interpretation report.

## Saving the 12-Lead Interpretation Report

At the completion of 12-lead ECG interpretation, select **Save** to save the report, these can then be viewed in **Review**.



## Printing the 12-Lead Interpretation Report

At the completion of 12-lead ECG interpretation, select **Print** or **Record** to output the report via the printer or recorder.

## Exiting the ECG 12-Lead Screen

To exit the ECG 12-Lead screen, select **Exit** on the ECG 12-Lead screen.

These guidelines do NOT replace the instructions for use and all users should refer to the appropriate Operator's Manual for detailed instructions.  
The Illustrations may not necessarily reflect the setup or data displayed on your device.