


# CO<sub>2</sub> Module Measure & Standby modes Quick Guide

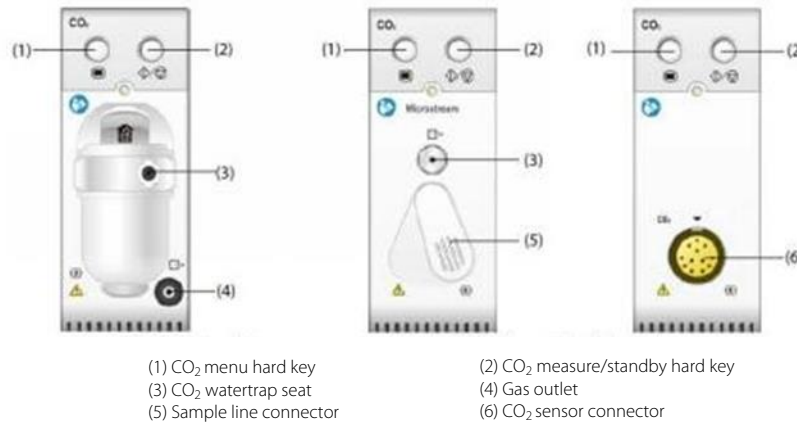

## To enter or exit the Standby mode manually

- Select the hard key on the CO<sub>2</sub> module to enter or exit **Measure / Standby** mode
- Select **Operating Mode** in the **CO<sub>2</sub> Setup** menu and toggle between **Measure** and **Standby**

## Information displayed when the CO<sub>2</sub> Module is in Standby mode



When the CO<sub>2</sub> module is placed in standby mode, the sample intake pump sets to a sample flow rate of zero and the word **Standby** appears on the monitor display

When exiting **Standby** mode by pressing the , the CO<sub>2</sub> intake pump defaults to the preset flow rate

**Please Note:** The system configuration may be set to automatically place the CO<sub>2</sub> module into standby mode following a predetermined period of inactivity